DEVILBISS®

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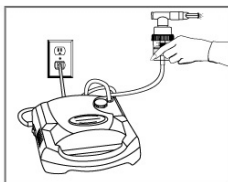
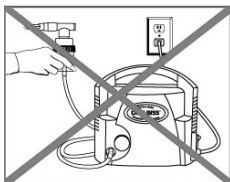
 **SUNRISE
MEDICAL®**

ENGLISH

IEC Symbols	3
Important Safeguards	3
Introduction	4
Important Parts of Your DeVilbiss Compressor	5
Important Parts of Your Disposable Nebulizer	5
Important Parts of Your Reusable Nebulizer	5
How to Operate Your Compressor.....	6
Nebulizer Cleaning	8
Compressor Cleaning	9
Filter Change	9
Maintenance	9
Specifications	10
Warranty.....	10
Declaration of Conformity	11

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FUSE RATING

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

DANGER— Urgent safety information for hazards that will cause serious injury or death.

WARNING— Important safety information for hazards that might cause serious injury.

CAUTION— Information for preventing damage to the product.

NOTE— Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

DANGER

To reduce the risk of electrocution:

1. Always unplug this product immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product that has fallen into water. Unplug immediately.

WARNING

To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if:
 - a. It has a damaged power cord or plug.
 - b. It is not working properly.
 - c. It has been dropped or damaged.
 - d. It has been dropped into water.

Return the product to an authorized Sunrise service center for examination and repair.

5. Keep the power cord away from heated surfaces.
6. Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
7. Never use while drowsy or asleep.
8. Never drop or insert any object into any opening or hose.
9. Do not use outdoors. This product is for household use only.
10. Do not use in an oxygen enriched environment.
11. Do not use this product with a DC to AC converter, or with any AC voltage and/or frequency other than specified.
12. Do not operate unit in a backpack or carrying bag.

NOTE– This unit employs both a temperature and current limiting device that will shut off the entire product should a malfunction occur or in the event of product misuse, abuse, or abnormal operation. Contact your qualified Sunrise provider for a replacement unit.

If replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

SAVE THESE INSTRUCTIONS

INTRODUCTION

Your doctor has prescribed a liquid medication to treat your respiratory condition. To best utilize this liquid medication, he has prescribed a DeVilbiss brand compressor/nebulizer. Your DeVilbiss compressor/nebulizer works to convert the medication to a high-quality mist of fine particles that penetrates deep into the lungs.

Be sure that you read and understand the information in this instruction guide. By following these simple instructions and the advice of your physician, your compressor will become an effective addition to your therapeutic routine.

Statement of Intended Use

The DeVilbiss compressor/nebulizer Model 3655 includes an AC powered air compressor that provides a source of compressed air for home health care use. The compressor is used in conjunction with a jet (pneumatic) nebulizer to convert certain inhalable drugs into an aerosol form for inhalation by a patient. The device can be used with adult or pediatric patients.

NOTE– If using a nebulizer other than one distributed by Sunrise Medical, verify that the pressure and/or flow characteristics of the nebulizer (while driven by the Model 3655) meet the nebulizer manufacturer's recommendation.

IMPORTANT PARTS OF YOUR DEVILBISS COMPRESSOR (Figures A & B)

1. Switch/Power Connector
2. Filter (non-washable)
3. Air-outlet connector
4. Carrying handle
5. Power cord storage recess

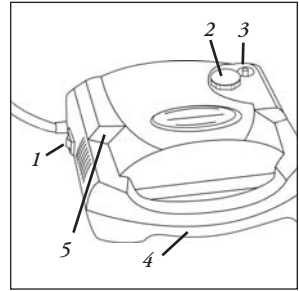
Figure A

IMPORTANT PARTS OF YOUR DISPOSABLE NEBULIZER (Included w/Models 3655A & 3655SE) (Figure C)

1. Mouthpiece
2. T-piece
3. Cap
4. Medication Cup
5. Nebulizer Air-Inlet Connector
6. Baffle
7. Tubing

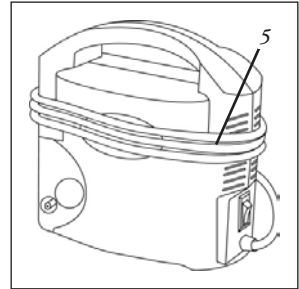
Figure B

Nebulizer parts are not sold separately. Additional nebulizers (part #4650D-621) are available through your Sunrise provider.

IMPORTANT PARTS OF YOUR REUSABLE NEBULIZER (Included w/Model 3655U) (Figure D)

1. Mouthpiece
2. Cap
3. Baffle
4. Medication Cup
5. Nebulizer Air-Inlet Connector
6. Interrupter Adapter (if applicable)

Nebulizer parts are not sold separately. Additional nebulizers (Part #s 800D/800C/800G) are available through your Sunrise provider.

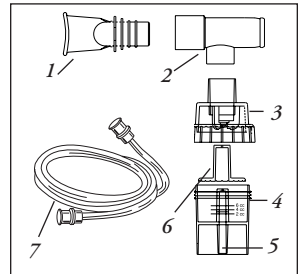
Figure C - Disposable Nebulizer

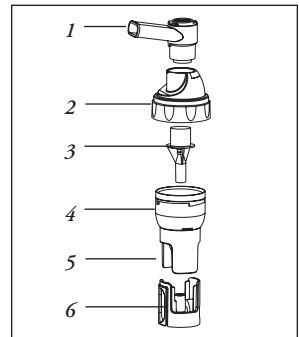
Figure D - Reusable Nebulizer

HOW TO OPERATE YOUR COMPRESSOR

NOTE— Before initial operation, your nebulizer should be cleaned by following the cleaning instructions, or as recommended by your physician or Sunrise provider.

1. Make sure the power switch is in the “Off” (O) position (**Fig. 1**).
2. Unwrap power cord.
3. Place compressor so that the rubber feet are resting on a level, sturdy surface (**Fig. 2**) and upon which the controls can be easily reached when seated. Never block the air openings of the product or place it on a soft surface, such as a patient’s lap or a bed or couch where the air openings may be blocked.
4. Plug power cord into an appropriate wall outlet.

⚠ DANGER

DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.

5. Wash hands.
6. Connect one end of the nebulizer tubing to the compressor air-outlet connector (**Fig. 2**).

NOTE—During high-humidity weather conditions, condensation (water build up) can occur in the nebulizer tubing.

7. Assemble clean nebulizer parts by placing baffle down in medication cup. Add prescribed medication into the medication cup (**Fig. 3**). Holding cup stationary, screw on nebulizer cap (**Fig 4**).
8. Assemble mouthpiece and t-piece (if applicable) and insert into the top of the nebulizer cap. If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.

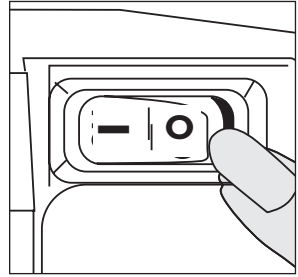
Figure 1

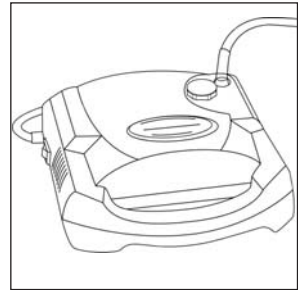
Figure 2

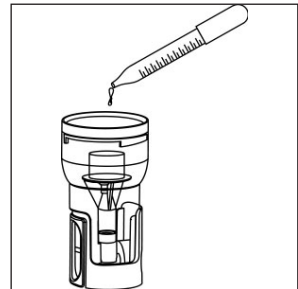
Figure 3

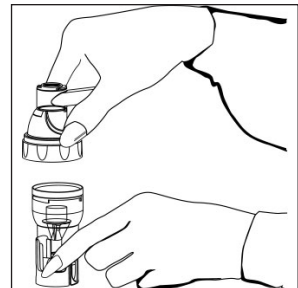
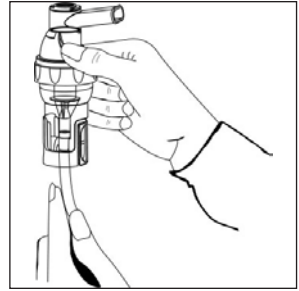
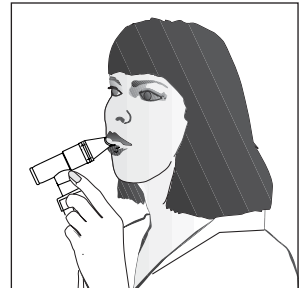
Figure 4

9. Attach tubing to nebulizer air-inlet connector (**Fig. 5**).
10. Press the power switch "On" (I) to start the compressor.
11. Begin treatment by placing the mouthpiece between teeth. With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow, hold for 2-3 seconds, then exhale slowly through the mouthpiece (**Fig. 6**). If treatment needs to be interrupted, simply press power switch "Off" (O).

*Figure 5*

NOTE—Your physician may recommend a specific breathing pattern during treatment. If so, follow his/her recommendations.

12. If an aerosol mask is used, place the mask over mouth and nose (**Fig. 7**). As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.
13. When treatment is complete, turn unit off by pressing the power switch to the "Off" (O) position. Unplug the unit from the wall power outlet.

*Figure 6**Figure 7*

NEBULIZER CLEANING

All parts of the nebulizer, except tubing, should be cleaned according to the following instructions. Your physician and/or Sunrise provider may specify a certain cleaning procedure. If so, follow their recommendations.

⚠ WARNING

To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment. Disinfecting is recommended once a day.

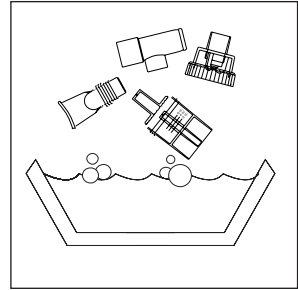
Figure 8

Clean After Every Use:

1. With power switch in the "Off" (O) position, unplug power cord from wall outlet.
2. Disconnect tubing from the air-inlet connector and set aside.
3. Disassemble mouthpiece or mask from cap. Open nebulizer by turning cap counterclockwise and removing baffle.
4. Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue. Allow to air dry.
5. For reusable nebulizer only, clean daily in dishwasher using top shelf.

CAUTION– The DeVilbiss Reusable Nebulizer is dishwasher safe, but do not place any disposable nebulizer parts in an automatic dishwasher; doing so could cause damage.

Disinfect Daily:

1. Using a clean container or bowl, soak items in three parts hot water to one part white vinegar for 30 minutes (**Fig. 8**) OR use a medical bacterial-germicidal disinfectant available through your provider. Be sure to follow manufacturer's instructions carefully.
2. With clean hands, remove items from disinfectant solution, rinse under hot tap water, and air dry on a clean paper towel. Store in a zip-lock bag.

NOTE–Do not towel dry nebulizer parts; this could cause contamination.

⚠ WARNING

To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

3. Keep the outer surface of the tubing dust-free by wiping regularly. Nebulizer tubing does not have to be washed because only filtered air passes through it.

NOTE– The DeVilbiss Disposable Nebulizer will last at least 15 days and possibly longer, depending on usage. The DeVilbiss Reusable Nebulizer will last up to one year. Proper cleaning will also help extend the life of the nebulizers. We recommend that an extra nebulizer be kept on hand at all times.

COMPRESSOR CLEANING

1. With power switch in the "Off" (O) position, unplug power cord from wall outlet.
2. Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

DANGER

Do not submerge in water; doing so will result in compressor damage.

FILTER CHANGE

1. Filter should be changed every 6 months or sooner if filter discolours.
2. Remove filter (**Fig. A**) by grasping it firmly and pulling out of the unit. Discard old filter.
3. Replace with a new DeVilbiss filter (part #3655D-601). Additional filters should be purchased from your Sunrise provider.
4. Push new filter into position.

CAUTION– Do not reuse or wash filter or substitute any other material such as cotton for a DeVilbiss air-inlet filter. It will result in compressor damage.

MAINTENANCE

All maintenance must be performed by a qualified Sunrise provider or authorized service center.

DANGER

Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified Sunrise provider.

SPECIFICATIONS

Size.....7.5" W x 4.0" H x 7.2" D
(19.1 cm x 10.2 cm x 18.3 cm)

Weight4.8 lbs. (2.18 kg)

Electrical

Requirements 3655A1.4 amps, 240 VAC (±10%) 50 Hz
3655SE & 3655U1.4 amps, 230 VAC (±10%) 50 Hz

Sound Level.....56 dBA

Power Consumption160 watts maximum

Max. Compressor Pressure.....35 psig or greater* (241 kPa)

Compressor Free Air Flow8 lpm or greater*

Operating Temperature Range+40° to +104°F
(+5 to +40°C)

Operating Humidityup to 95% non-condensing

Storage/Transport Temperature Range-40° to +158°F
(-40° to +70°C)

Storage/Transport Humidityup to 95% non-condensing

Supplied Nebulizer Capacity6 ml (cc)

Supplied Nebulizer Nebulization Rate0.15 ml/min or greater*

Supplied Nebulizer MMAD5 microns or less**

*Conditions may vary based on altitude above sea level, barometric pressure and temperature.

**Value determined with minimum flow rate through the nebulizer (20° C, sea level).

This product complies with the following electromagnetic compatibility standard: IEC 60601-1-2:2nd Edition

Equipment Classifications:

Protection against electric shock:.....Class II

Degree of protection against electric shock:Type BF

Equipment not suitable for use in the presence of flammable anaesthetic mixture with air or with oxygen or nitrous oxide

Mode of operation:.....Continuous operation

FIVE-YEAR LIMITED WARRANTY ON COMPRESSOR PARTS

All DeVilbiss Compact Compressor models are warranted to be free from defective workmanship and material for a period of five years from the date of purchase. Any defective part(s) will be repaired or replaced at Sunrise’s option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your authorized Sunrise provider or Sunrise Service Department at 800-333-4000 (814-443-4881) (Europe 44-138-444-6688) for instructions. NOTE— This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

DECLARATION OF CONFORMITY

Manufacturer: Sunrise Medical
Address: Respiratory Products Division
100 DeVilbiss Drive
Somerset, Pennsylvania 15501-2125 USA

Product Designation: Compressor/Nebulizer

Type, Model: DeVilbiss® 3655SE & 3655U

We herewith declare that the above-mentioned product complies with the requirements of EC Directive 93/42/EEC and the following:

Quality System Standards Applied: ISO9001/ISO13485

Notified Body RWTÜV

MDD Annex II Applied

Safety Standards Applied: IEC60601-1:1988 including A1 & A2

EMC Compliance To: EN60601-1-2 (2001)

European Contact: Sunrise Medical Ltd.
Sunrise Business Park
High Street
Wollaston, West Midlands DY8 4PS
ENGLAND
44-138-444-6688

®Registered U.S. Patent and Trademark Office and other countries.

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MANUFACTURER'S NOTE

Thank you for choosing a DeVilbiss compressor/nebulizer. We want you to be a satisfied customer. If you have any questions or comments, please send them to our address on the back of this manual.

FOR SERVICE CALL YOUR AUTHORIZED SUNRISE PROVIDER:

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237 Romina Drive, Unit 3
Concord, Ontario L4K 4V3
CANADA
905-660-2459

Sunrise Medical Ltd.

Sunrise Business Park
High Street
Wollaston, West Midlands DY8 4PS
ENGLAND
44-138-444-6688

Sunrise Medical Pty. Limited

15 Carrington Road, Unit 7
Castle Hill NSW 2154
AUSTRALIA
61-2-9899-3144

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Printed in the USA
on recycled paper.

©Sunrise Medical 02.04

SE-655 Rev. B