

If you need to get up throughout the night, and don't want to have to repeat this process, remember to use our detachable swivel port. By pressing in the side clips on the swivel it can be disconnected and reconnected easily.



MAKERS OF RESPIRATORY VALVES SINCE 1938

HANS RUDOLPH, inc.

www.rudolphkc.com

TEL: (913) 422-7788

U.S.A. & CANADA: (800) 456-6695

FAX: (913) 422-3337


E-Mail: hri@rudolphkc.com

8325 COLE PARKWAY, SHAWNEE, KANSAS 66227 U.S.A.

Full Face CPAP Mask

V2™ Fitting Instructions


New V2™ Features

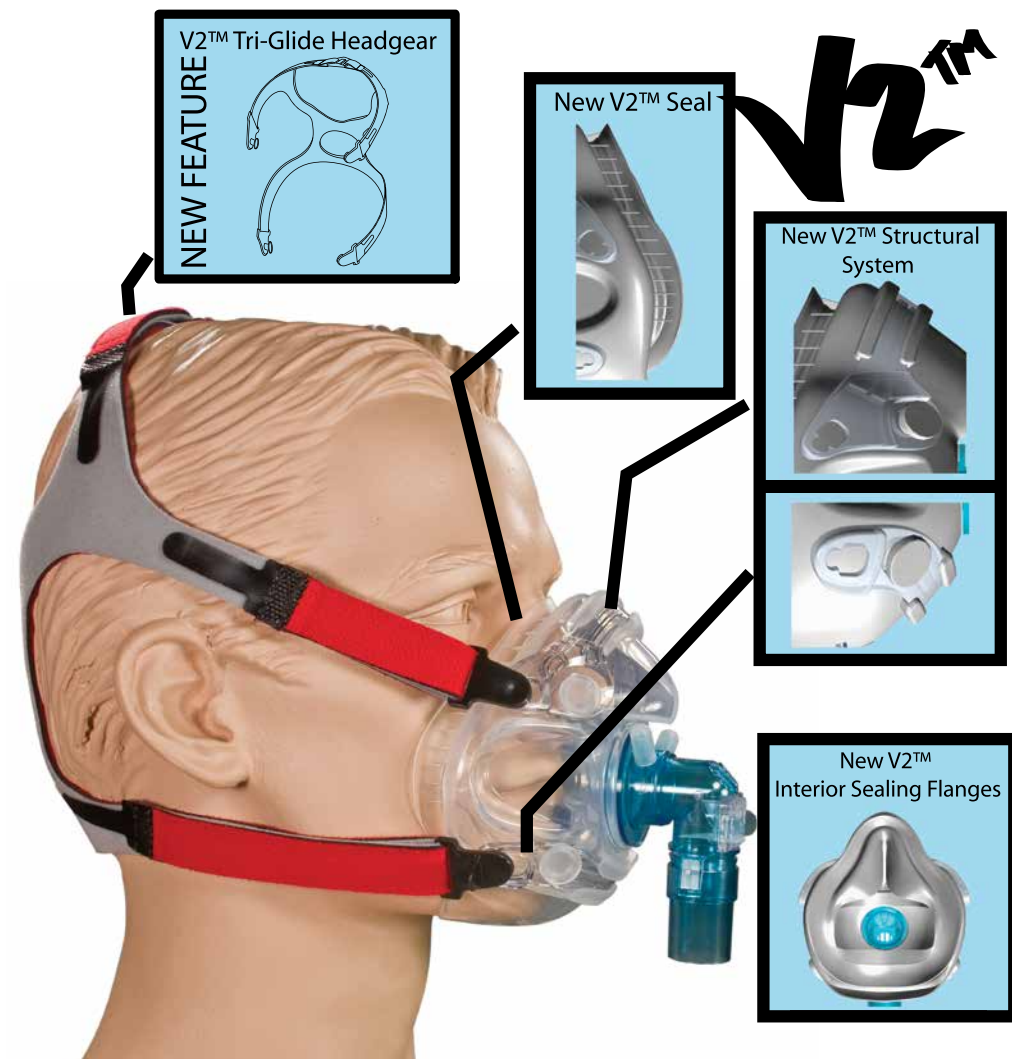

 **MultiDoctorShop**
 Medical products at home


We provide the official document of the manufacturer

www.multidoctorshop.com

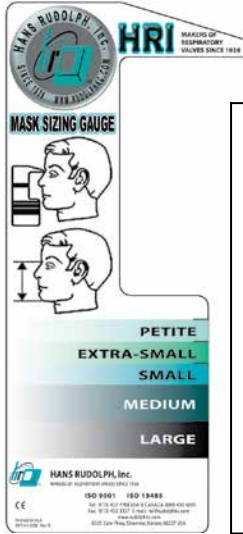

First, use our mask sizing gauge to help determine which size would be best for you. Remember that our mask runs larger than others on the market. In some cases patients have gone a size and even two sizes smaller than what they have worn in another brand. We go larger to ensure fits on a greater variety of facial types.

Secondly, take the V2 in your hands and place it comfortably onto your face. You may want to disconnect one of the lower headgear clips at this point.

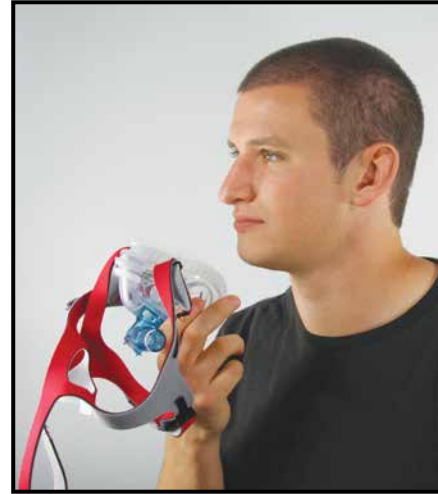

Then grip the headgear's lowest section and pull it up & over to fit onto the crown of your head. This can be done with all straps connected, but disconnecting one of the lower headgear clips makes this task much easier.



Once the headgear is in place reconnect the lower headgear clip (if it was disconnected).



Now, adjustments to the headgear can be made to create the seal and the feel that you are looking for. Both the upper and lower straps can be tightened by pulling the Velcro straps out and back.



The strap on top of your head can be tightened or loosened by adjusting the Tri-Glide connection. Adjustments to this top strap connection can be made by pulling the Velcro straps up and then pulling outward.



Tri-Glide Feature Added to headgear.



Further adjustment can be made even while hooked up to the blower. These techniques can sometimes help resolve a leak problem without having to take the mask off or even turn the air off. The V2™ Mask is pliable so don't be afraid to pull on the sides and bottom of the mask to adjust the position of the seal.