

SALTER LABS®

Salter-Style® High Flow Adult Cannula

"The Cannula that acts like a Mask"

Performance, comfort and long term wearability of this cannula virtually eliminates the need for masks to deliver high flow oxygen (up to 15 LPM).

Salter high flow cannulas maintain adequate oxygen saturation for those special high flow patients without using a mask. It increases patient comfort by allowing the patient to eat, drink, talk and take oral medications including nebulizer treatments without assistance.

End result can be a reduction in care giving costs.



Features

- Delivers oxygen flows up to 15 LPM
- Special three channel headset tubing with enhanced reservoir facepiece
- Unique, rectangular style headset tubing slide bolo
- Seamless, extra soft, reservoir facepiece anatomically contoured to fit upper lip
- Bright green supply tubing from the cannula to oxygen source

Benefits

- Virtually eliminates the need to use face masks to achieve desired O₂ saturation
- Improves cannula high flow performance with quieter operation
- Allows easy, secure adjustment and fit for optimum patient comfort
- Provides long term wearability and patient comfort
- Visually alerts attending care givers that a high flow cannula is in use

Can be used in hospital, hospice, nursing home and home care environments




slc 09

Salter-Style® Adult High Flow Cannula 1600HF

The Salter-Style® Adult High Flow Cannula (#1600HF) features special larger bore safety channel headset tubing with an enhanced reservoir facepiece. This allows effective delivery of oxygen flows up to 15 LPM. It incorporates all of the unique design features that have made Salter-Style® cannulas the worldwide standard for comfort and efficiency. 1600HF Cannulas are anatomically shaped with soft, curved nasal prongs to provide genuine long-term “wearability comfort”. Bright green supply tubing alerts attending personnel that a high LPM flow oxygen cannula is in use.

Professional Instructions for Use

The Salter-Style® Adult High Flow Cannula has been extensively lab tested to deliver oxygen flows up to 15 LPM. Clinical results obtained in individual patients may vary. It is recommended that, as with any oxygen delivery device, patients be individually tested to titrate dosing and to determine the suitability of this device to meet their unique and varying individual needs and dosage requirements.

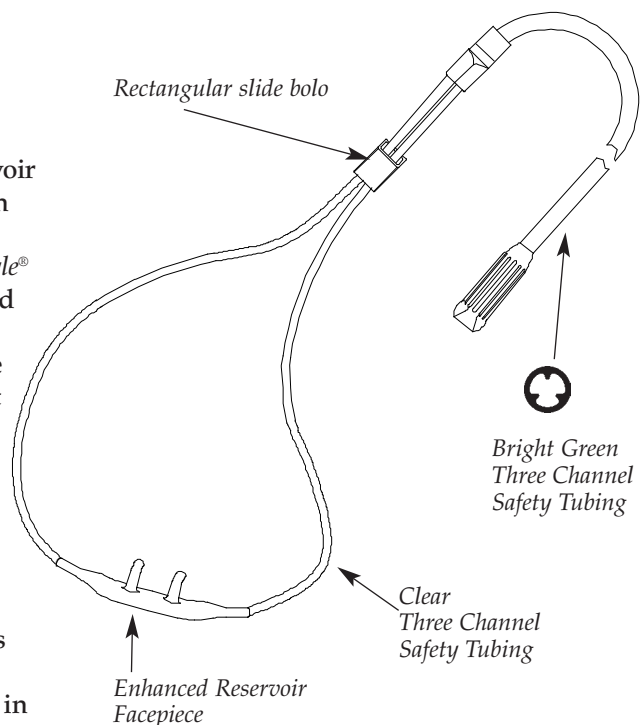

Ordering Information

	Units	Part (per case)	Number
<i>Adult High Flow Cannula—For oxygen flows up to 15 LPM</i>			
<i>Salter-Style® Cannulas</i>			
Adult Clear cannula with enhanced reservoir facepiece, and a 7' Bright Green supply tubing	25		1600HF*
Adult Clear cannula with enhanced reservoir facepiece, and a 4' Bright Green supply tubing	25		1600HF-4*
Adult Clear cannula with enhanced reservoir facepiece, and a 14' Bright Green supply tubing	25		1600HF-14*
Adult Clear cannula with enhanced reservoir facepiece, and a 25' Bright Green supply tubing	10		1600HF-25*
Adult Clear cannula with enhanced reservoir facepiece, and a 50' Bright Green supply tubing	10		1600HF-50*



100 W. Sycamore Road - Arvin, CA 93203
 Phone: 661-854-3166
 1-800-421-0024
 1-800-235-4203
 Fax: 661-854-3850
 Toll Free Fax: 1-800-628-4690
 E-mail: salterlabs@us.salterlabs.com
 www.salterlabs.com
 ISO 13485

CE
0482