

F&P Brevida™



NASAL PILLOWS MASK

Designed for Confidence. Built for Comfort.

NEW

Gaining patient confidence is key to successful CPAP therapy and patient confidence begins with a mask that fits and is comfortable.



 MultiDoctorShop
Medical products at home

We provide the official
document of the manufacturer
www.multidoctorshop.com

Fisher & Paykel
HEALTHCARE

F&P Brevida™ Key features and benefits

Adjustable Headgear

Adjustable headgear which is designed to stay in place during sleep.

AirPillow™ Seal

The innovative AirPillow™ inflates to form a gentle and effective seal.




VisiBlue™

Blue highlights incorporated into the mask key components assist patients with daily mask use.

Washable Diffuser

The diffuser, which reduces noise and air draft, is washable and durable.

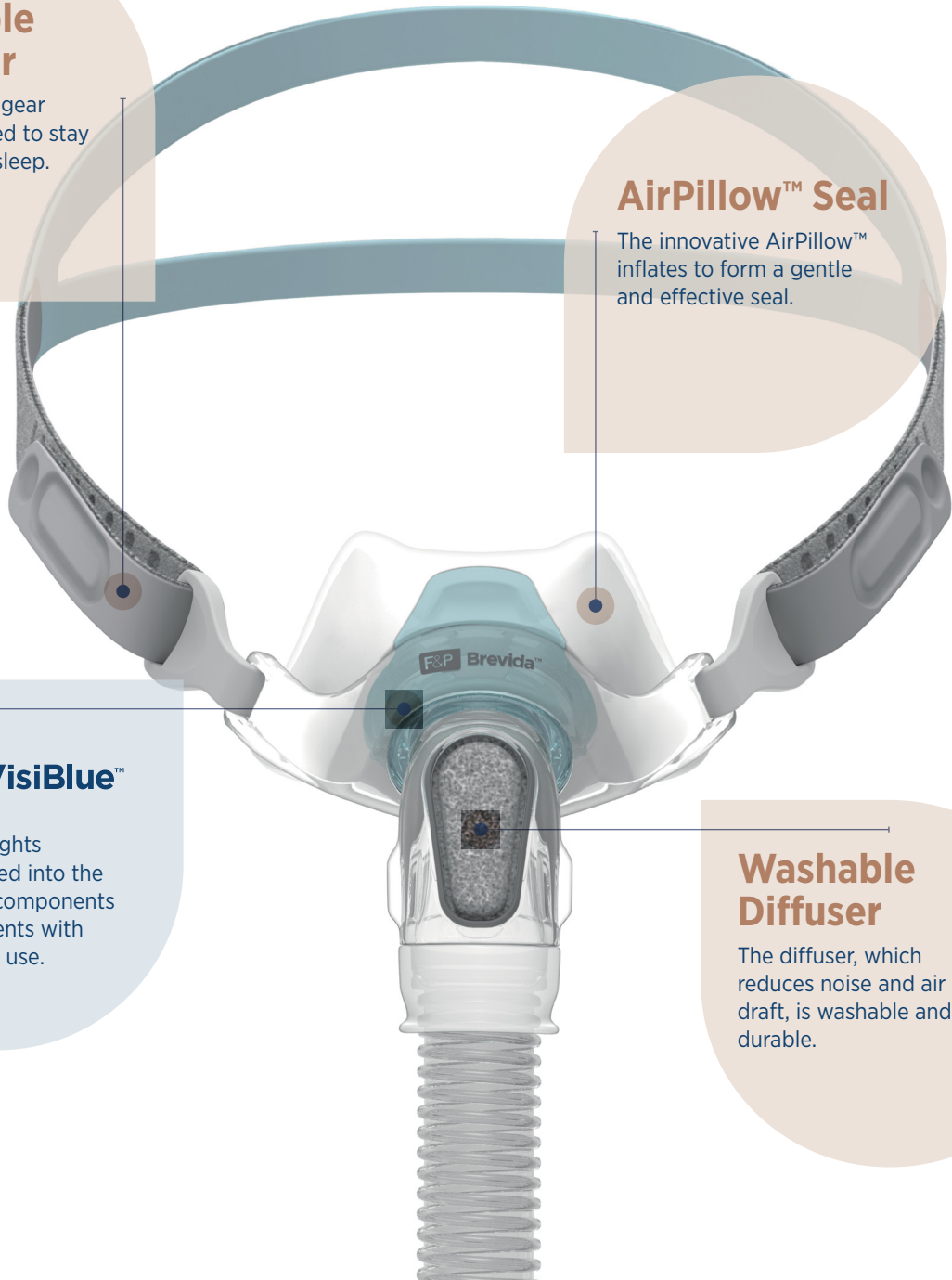

The innovative F&P Brevida™ AirPillow™ Seal

Two seal sizes

Extra Small-Small and Medium-Large.

Extra-thin, soft seal


Sealing region made of extra-thin (0.25 mm) soft silicone to provide a gentle seal.

Versatile pillows

Versatile pillow geometry to fit a wide range of nostril sizes.

Stabilizing side supports

Side supports designed to reduce the likelihood of the seal dislodging during sleep.

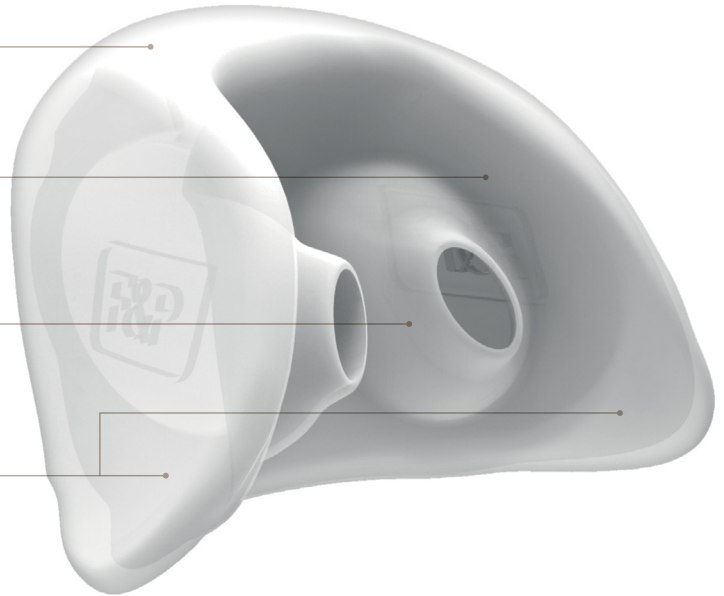


A gentle, soft seal

The AirPillow™ seal inflates to form a gentle ‘pillow’ of air and seals in and around the nose.


F&P AirPillow™ Nasal Pillows Seal

Seals in and around
the nose

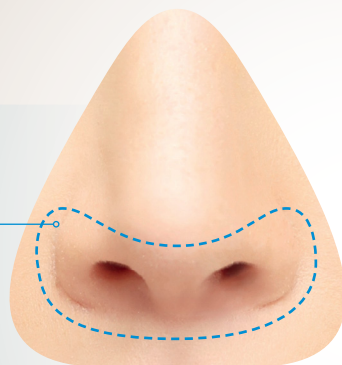

Traditional Nasal Pillows Seal

Seals directly in the
nostrils

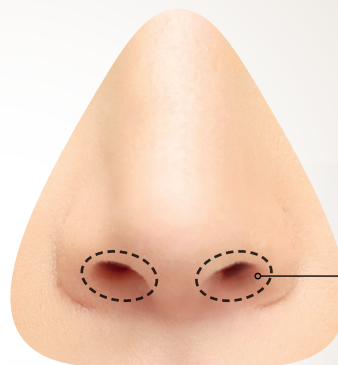

Fitting your F&P Brevida™



01

Step 1.
Hold the silicone seal away from your nose and use your fingers to carefully spread open the soft side supports of the silicone seal and then guide the two openings into your nostrils.



02

Step 2.
Hold the mask frame in position with one hand and pull the headgear over your head with the other hand, positioning the straps above the ears as per the fitting image above.



03

Step 3.
Gently tighten or loosen the headgear straps on the sides of the headgear until you achieve a comfortable seal.

REORDER CODES

MASKS

BREISA Brevida Extra Small-Small (one XS-S seal)

BREIMA Brevida Medium-Large (one M-L seal)

BREISMA Brevida XS-S and M-L (two seals)

SPARE PARTS

400BRE113 Brevida Seal Extra Small-Small

400BRE114 Brevida Seal Medium-Large

400BRE120 Brevida Headgear

400BRE151 Brevida Headgear Clips

400BRE161 Brevida Diffuser

400BRE171 Brevida Elbow

400FPH241 F&P Swivel 2

400BRE131 Brevida Mask No Headgear XS-S

400BRE132 Brevida Mask No Headgear M-L