

# DreamWisp patient fitting guide

Follow the instructions to get started fitting your DreamWisp mask



1 Select the cushion that fits the width of your nose without blocking your nostrils. The top of the cushion may sit between the bridge and the tip of your nose. There are five cushion sizes available: petite, small, medium, large, extra-large




2 Unclasp one or both of the magnetic clips from the magnetic clip receiver.




3 Pull the headgear over the top of your head.




4 Place the cushion over your nose.

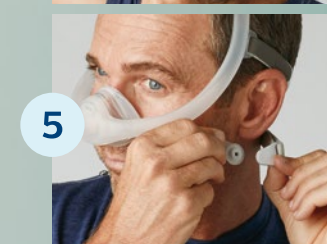

5 Clasp the magnetic clips to the magnetic clips receivers.

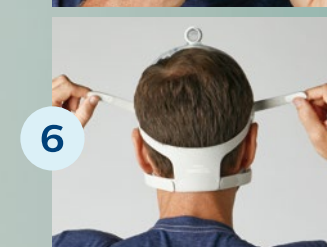

6 To adjust the headgear, peel the bottom headgear tabs away from the fabric. Evenly adjust the length of the straps. Press the tabs back against the fabric to reattach. Repeat this step with the top straps.

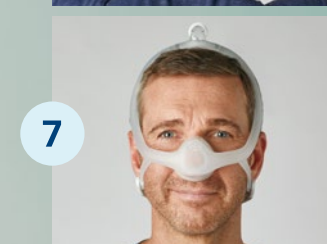

7 Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head. The cushion should rest comfortably on your face without blocking your nostrils.

**Note:** Please read the Instructions for use for further fitting information.

 MultiDoctorShop

Medical products at home

We provide the official document of the manufacturer

[www.multidoctorshop.com](http://www.multidoctorshop.com)