


## Simply Fits + Performs

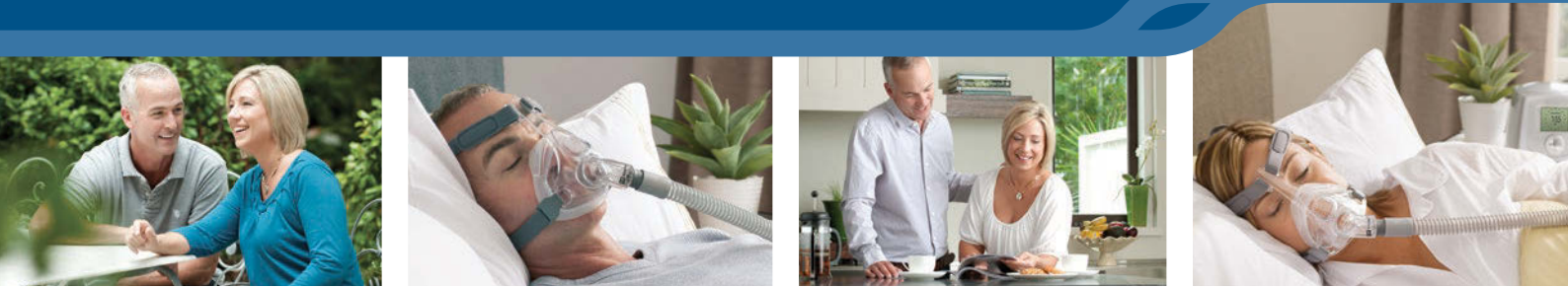

### F&P SIMPLUS FULL FACE MASK

When engineering the F&P Simplus our designers set out to create a mask that revolutionized Full Face comfort, seal and easy use.

The F&P Simplus is the result of intensive research and development, many prototypes and thousands of design hours. We consulted patients, equipment providers and clinicians from around the world to develop a mask that **simply fits + performs.**

## F&P Simplus Features

The F&P Simplus incorporates three key components, the **RollFit™ Seal**, **ErgoForm™ Headgear** and **Easy Frame**, all designed to work in harmony. In combination, these components offer the comfort, seal and easy use that Fisher & Paykel Healthcare masks are known for.

**ErgoForm Headgear**  
Easy-Clip Hooks +  
Adjustable Crown Strap +  
Stretch and Non-stretch Panels +  
Embedded VELCRO® Tabs +

**RollFit Seal**  
Integrated Stability Panels +  
Advanced Air Diffuser +

**Easy Frame**  
Easy-Clip Frame Attachment +  
Ball-and-Socket Elbow +



### 1. RollFit Seal

As the name suggests, the one-piece seal “rolls” back and forth on the bridge of the nose to adjust automatically. In doing so, the **RollFit Technology** reduces pressure on the bridge of the nose.

The **RollFit Seal** incorporates **Integrated Stability Panels** to enhance the sealing performance and the **Advanced Air Diffuser** is designed for minimal noise and draft.

### 2. ErgoForm Headgear

This breathable, ergonomically designed headgear self-locates high on the rear of the head, allowing for unrestricted head movement (sideways, up and down).

Incorporating **Stretch and Non-stretch Panels**, the **ErgoForm Headgear** is designed to hold its shape and allow stretch where it is required, for optimal usability and performance.

**Embedded VELCRO®\* Tabs** with **Easy-Clip Hooks** enable fast fitting and removal.

The **Adjustable Crown Strap** enhances mask stability.

### 3. Easy Frame

This low-profile frame is stable, durable and small, and ensures a clear line of sight.

The one frame fits all three seal sizes and has an **Easy-Clip Frame Attachment** to aid assembly after cleaning.

The **Ball-and-Socket Elbow** rotates for freedom of movement while reducing drag from the CPAP tube.

#### Mask Reorder Codes

<b>400475</b>	F&P Simplus Mask – Small
<b>400476</b>	F&P Simplus Mask – Medium
<b>400477</b>	F&P Simplus Mask – Large

#### Spare Parts Reorder Codes

<b>400HC579</b>	F&P Simplus Seal – Small	<b>400HC582</b>	F&P Simplus Headgear – Small
<b>400HC580</b>	F&P Simplus Seal – Medium	<b>400HC583</b>	F&P Simplus Headgear – Medium/Large
<b>400HC581</b>	F&P Simplus Seal – Large	<b>400HC584</b>	F&P Simplus Mask No H/G – Small**
<b>400HC570</b>	F&P Simplus Swivel	<b>400HC585</b>	F&P Simplus Mask No H/G – Medium**
<b>400HC587</b>	F&P Simplus Headgear – Clips & Buckle	<b>400HC586</b>	F&P Simplus Mask No H/G – Large**

\* VELCRO® is a registered trademark of Velcro Industries B.V.  
\*\* Not available in all countries