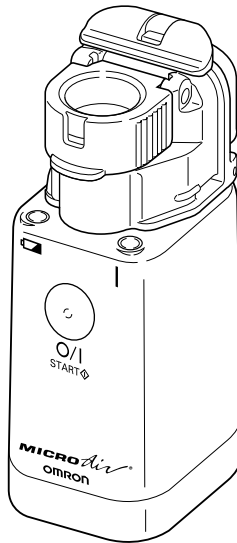
# OMRON

## INSTRUCTION MANUAL

# MICRO *Air*®

## VIBRATING MESH NEBULIZER

### Model NE-U22V

 MultiDoctorShop  
Medical products at home  
We provide the official  
document of the manufacturer  
[www.multidoctorshop.com](http://www.multidoctorshop.com)

English

Français

Español

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician and/or licensed healthcare practitioner.

# TABLE OF CONTENTS

## **Before Using the Device**

|   |   |
|---|---|
| Introduction .....                      | 3 |
| Safety Information .....                | 4 |
| Know Your Unit .....                    | 7 |
| Accessories and Replacement Parts ..... | 8 |
| Unit Assembly.....                      | 9 |

## **Operating Instructions**

|                                       |    |
|---------------------------------------|----|
| Battery Installation .....            | 11 |
| Using the AC Adapter .....            | 12 |
| Filling the Medication Bottle .....   | 13 |
| Selecting the Nebulization Mode ..... | 14 |
| Using the Device .....                | 15 |

## **Care and Maintenance**

|                              |    |
|------------------------------|----|
| Cleaning after Each Use..... | 16 |
| Daily Disinfecting .....     | 17 |
| Caring for the Device .....  | 18 |

## **Troubleshooting**

|                             |    |
|-----------------------------|----|
| Troubleshooting Guide ..... | 20 |
| Warranty Information .....  | 22 |
| FCC Statement .....         | 23 |
| Specifications .....        | 24 |

# INTRODUCTION

Thank you for purchasing the Omron® NE-U22V MICROAir® Vibrating Mesh Nebulizer.

Fill-in for future reference

DATE PURCHASED: \_\_\_\_\_

SERIAL NUMBER: \_\_\_\_\_

Staple your purchase receipt here.

Omron Healthcare has moved forward in the development of electronic nebulizer technology with the introduction of the NE-U22V MICROAir® Vibrating Mesh Nebulizer. Focusing on patient convenience and compliance as its goal, this device offers ultimate portability for wherever you go and revolutionary vibrating mesh technology that provides a precise, powerful and effective treatment every time.

The MICROAir® is a vibrating mesh nebulizer system designed to aerosolize liquid medications for inhalation by the patient. The device may be used with pediatric and adult patients in the home, hospital, and sub-acute care settings.

Your NE-U22V MICROAir® comes with the following components:

- Main Unit
- Unit Cover
- Medication Bottle
- Mesh Cap
- Mask and Mouthpiece Adapter
- Mouthpiece
- Storage Case
- Instruction Manual
- Instructional DVD

The following are optional accessories sold separately:

- AC Adapter
- Child Mask

*The MICROAir® is a prescription medical device. Operate this device only as instructed by your physician and/or licensed healthcare practitioner.*

**SAVE THESE INSTRUCTIONS**

# SAFETY INFORMATION

To assure the correct use of the product basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

## SAFETY ICONS USED IN THIS INSTRUCTION MANUAL

|  |  |
|--|--|
|  <b>WARNING</b> | Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury   |
|  <b>CAUTION</b> | Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. |

## OPERATING THE DEVICE

-  Read all the information in the instruction book and any other literature included in the box before using the unit.
-  For type, dose, and regimen of medication follow the instructions of your physician and/or licenced healthcare practitioner.
-  Pentamidine is not an approved medication for use with this device.
-  Do not use tap or mineral water in the nebulizer for nebulizing purposes.
-  Clean and disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece and Mask and Mouthpiece Adapter before using the device for the first time after purchase.
-  If the device has not been used for a long period of time, clean and disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter before using them.
-  Always dispose of any remaining medication in the medication cup after each use. Use fresh medication each time you use the device.
-  Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months, or where it will be exposed to direct sunlight.

# IMPORTANT SAFETY NOTES

## OPERATING THE DEVICE (continued)

-  Provide close supervision when this device is used by, on, or near infants, children or compromised individuals.
-  Inspect the main unit and the nebulizer parts each time before using the device. Make sure no parts are damaged, the device is assembled properly, and the device operates normally.
-  To prevent damage to the device, add the medication slowly. Do not allow the medication to overflow the Medication Port.
-  Do not add more than 7 mL of medication to the medication cup.
-  To prevent damage to the device, make sure that the Mesh Cap is correctly in place. If the Mesh Cap is not properly closed, medication will leak.
-  Do not operate the device at temperatures greater than +104° F (+40° C).
-  Do not subject the device or any of the components to strong shocks, such as dropping on the floor.
-  This device is approved for human use only.
-  Do not disassemble or attempt to repair the device or components.
-  Operate the device only as intended. Do not use the device for any other purpose.
-  Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
-  Use only Omron authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
-  Changes or modifications not approved by Omron Healthcare will void the user warranty.

## RISK OF ELECTRICAL SHOCK WHEN USING THE AC ADAPTER

-  Do not plug or unplug the power cord into the electrical outlet with wet hands.
-  Use only the AC adapter designed by Omron for this device. Use of any other AC adapter may damage the device.

# IMPORTANT SAFETY NOTES

## RISK OF ELECTRICAL SHOCK WHEN USING THE AC ADAPTER (continued)

-  Do not overload power outlets. Plug the device into the appropriate voltage outlet.
-  Do not use extension cords. Plug the power cord directly into the electrical outlet.
-  Unplug the power cord from the electrical outlet after using the device.
-  Unplug the power cord from the electrical outlet before cleaning the device.

## MAINTENANCE AND STORAGE

-  Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed.
-  Do not immerse the main unit in water or other liquid.
-  Do not use or store the device in humid locations, such as a bathroom. Use the device within the operating temperature and humidity.
-  Do not leave the cleaning solution in the nebulizer parts. Rinse the nebulizer parts with distilled water after disinfecting.
-  Rinse the nebulizer parts after each use. Dry the parts immediately after washing.
-  Store the device and the components in a clean, safe location.
-  To prevent damage to the device, do not carry or leave the Medication Bottle filled with medication or distilled water.
-  Do not place or attempt to dry the device or any of its parts in a microwave oven.
-  To prevent damage to the device, do not rinse or immerse the Main Unit in any liquid, do not wash or rinse any of the parts under strong running water, and do not touch the mesh with your hand or any object.
-  Do not use household bleach. The mesh will rust and the Mesh Cap cannot be used.
-  To prevent damage to the device, do not clean the Main Unit using abrasive cleaners or any type of chemical, and do not allow any moisture to contact the electrodes or the AC Adapter jack on the Main Unit.
-  Do not put the AC Adapter in the Storage Case. The Storage Case is not intended to carry the AC Adapter.

# KNOW YOUR UNIT

## Main Unit

### Electrode

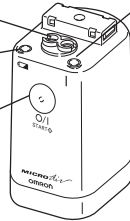
Power conductor from the Main Unit to the vibrator on the Medication Bottle

### Battery Low Indicator

An orange light blinks when the batteries are worn

### ON/OFF Button

Turns the power for the Main Unit on and off

### Power Indicator

The green light shows the power is on

### Bottom of Main Unit

### Battery Cover

Push lever to remove cover

### Electrode

Power conductor from the Main Unit to the AC Adapter Connection Stand

## Medicine Bottle

### Bottle Cap Locking Lever

Opens the Medication Bottle for cleaning

### Medication Bottle

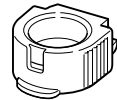
Holds maximum capacity of 7mL for treatment

### Medication Port

Add the medicine to the bottle

## Mesh Cap

Metal Alloy Mesh creates high efficiency aerosol

## Mask and Mouthpiece Adapter

Holds the Mouthpiece or Mask securely on the device

## Mouthpiece

Patient Interface

## Main Unit Cover

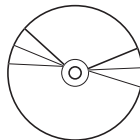
Protects the Main Unit with the attached Medication Bottle and Mesh Cap during storage

# KNOW YOUR UNIT

## Instruction Manual

## Instructional DVD


## Storage Case

Case holds the Main Unit,  
Medication Bottle, Mask and  
Mouthpiece Adapter

## Optional Accessory
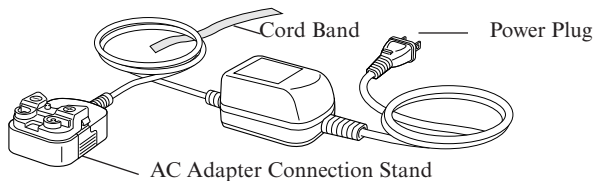
### AC Adapter Model No. U22-5

## Optional Accessory

### Child Mask Model No. C922

Patient Interface  
for pediatric use

## Replacement Parts

|                                | Model No. |
|--------------------------------|-----------|
| BATTERY COVER                  | U22-8     |
| MAIN UNIT COVER                | U22-9     |
| MASK AND MOUTHPIECE<br>ADAPTER | U22-2     |
| MEDICATION BOTTLE              | U22-3     |
| MESH CAP                       | U22-4     |
| MOUTHPIECE                     | U22-1     |
| STORAGE CASE                   | U22-7     |

# PREPARING THE NEBULIZER FOR USE

## WARNING

Read all the information in the instruction book and any other literature included in the box before using the unit.

## WARNING

Clean and disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece and Mask and Mouthpiece Adapter before using the device for the first time after purchase.

## WARNING

If the device has not been used for a long period of time, clean and disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter before using them.

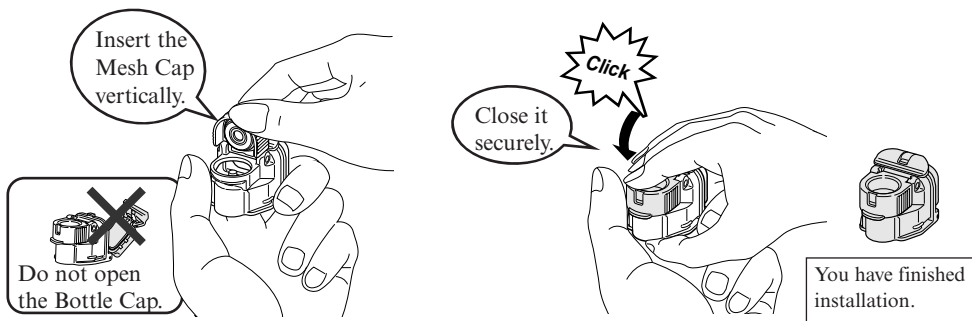
For directions on cleaning and disinfecting go to page 16 and 17 under Care and Maintenance.

**The device must be assembled before it is used.**

## General Information



- Components may fit tightly since they are made to prevent the medication from leaking.
- Hold the device securely with both hands.
- Install the parts securely. You may hear a click sound as you install some of the parts.

### 1. Attach the Mesh Cap to the Medication Bottle.


# PREPARING THE NEBULIZER FOR USE

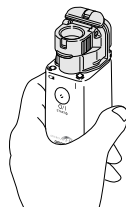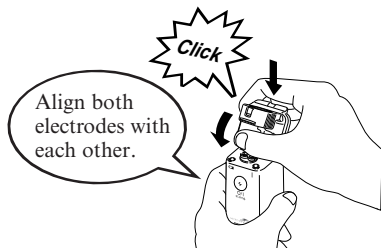2. Attach the Medication Bottle to the Main Unit.

You have finished installation.

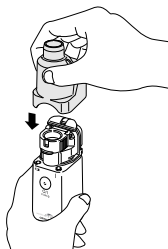
3. Attach the Mask and Mouthpiece Adapter to the Main Unit.


You have finished installation.

4. Attach the Mouthpiece or the Child Mask to the Mask and Mouthpiece Adapter.

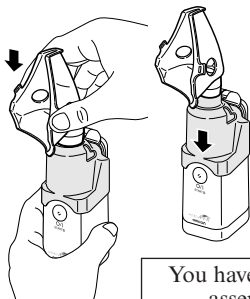- How to attach the Child Mask

- How to attach the Mouthpiece
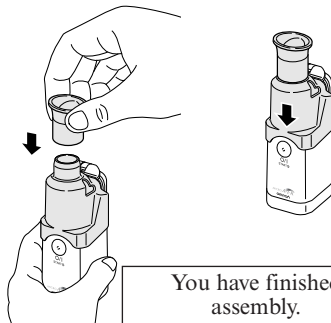
# BATTERY INSTALLATION

This device operates using two (2) AA alkaline batteries or two (2) NiMH AA rechargeable batteries.

## ⚠ CAUTION

Do not install worn and new batteries together.  
Do not use different types of batteries together.  
Remove the batteries if the device will not be used for three months or longer.

### 1. Remove the Battery Cover.

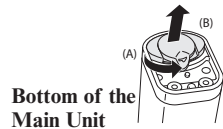
- Rotate the Battery Cover lever in the direction of the arrow as shown in the illustration.
- Remove the Battery Cover. The Battery Cover may appear to be tight fitting since it was designed to prevent fluids from getting into the device.

### 2. Insert the batteries.

Correctly align the polarities (+ and -) with the battery indication marks on the device.

### 3. Replace the Battery Cover.

Use your thumbs to push on both ends of the Battery Cover Press down firmly until you hear both tabs click into place.

Bottom of the Main Unit

## Battery Life and Battery Replacement

### Alkaline Batteries

- The device can be used for approximately 8 days if operating for 30 minutes a day.
- The Battery Low Indicator (orange light) flashes to signal the batteries are low. Replace both batteries with new ones.
- The Battery Low Indicator (orange light) turns on to signal the batteries are worn out. The device will not nebulize. Immediately replace both batteries with new ones.

### NiMH Rechargeable Batteries

- The device can be used for approximately 8 days if operating for 30 minutes a day when the batteries are fully charged.
- The Battery Low Indicator (orange light) flashes to signal the rechargeable batteries have little or no residual power remaining. If the device will not nebulize immediately recharge the batteries.
- Recharge the batteries using a commercially available battery charger suitable for the batteries used in the device.
- The AC Adapter does not function as a battery charger.

# USING THE AC ADAPTER

The device is designed not to draw power from the batteries when the AC Adapter is used. The AC Adapter is not a battery charger.

## WARNING

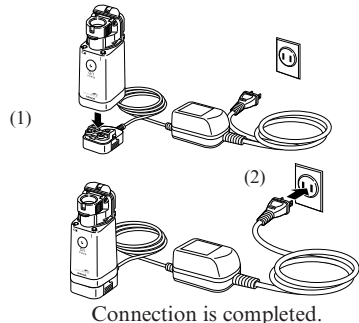
Use only the AC adapter designed by Omron for this device. Use of any other AC adapter may damage the device.

## To connect the AC Adapter to the Main Unit

- (1) Place the Main Unit on the AC Adapter Connection Stand as shown in the illustration below.

**NOTE:** It will click and lock to the stand.

- (2) Insert the AC Adapter Power Plug into a 120V electrical outlet.

## WARNING

Do not plug or unplug the power cord into the electrical outlet with wet hands.

## CAUTION

Do not overload power outlets. Plug the device into the appropriate voltage outlet.

## CAUTION

Do not use extension cords. Plug the power cord directly into the electrical outlet.

## CAUTION

Unplug the power cord from the electrical outlet before cleaning the device.

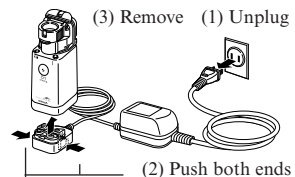
## To remove the AC Adapter from the Main Unit

- (1) Disconnect the AC Adapter Power Plug from the electrical outlet.
- (2) Push on both sides of the Connection Stand to unlock the Main Unit.
- (3) Remove the Main Unit

## CAUTION

Unplug the power cord from the electrical outlet after using the device.

# FILLING THE MEDICATION BOTTLE

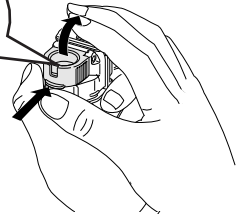
Remove the Mouthpiece or Mask and the Mask and Mouthpiece Adapter from the Main Unit.

## 1. Open the Mesh Cap.

Hold the device securely in your hand.

Lift up the mesh cap while pressing your thumb upward on the brim.

## 2. Fill the Medication Bottle.

### WARNING

Pentamidine is not an approved medication for use with this device.

### WARNING

Do not use tap or mineral water in the nebulizer for nebulizing purposes.

Be careful to prevent the Mesh Cap from closing as shown in the illustration.

The maximum capacity of the Medication Bottle is 7mL.

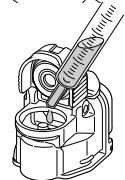
### CAUTION

Do not add more than 7 mL of medication to the medication cup.


### CAUTION

To prevent damage to the device, add the medication slowly. Do not allow the medication to overflow the medication port.

Check that the Mesh Cap is closed.
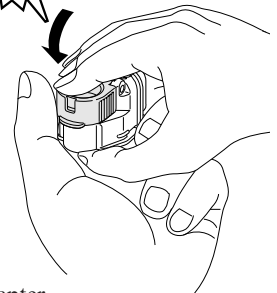
## 3. Close the Mesh Cap.

### CAUTION

To prevent damage to the device, make sure that the Mesh Cap is correctly in place. If the Mesh Cap is not properly closed, medication will leak.

## 4. Attach the Mask and Mouthpiece Adapter to the Main Unit

Attach the Mouthpiece or the Child Mask to the Mask and Mouthpiece Adapter.

**NOTE:** For instructions on attaching the Mask and Mouthpiece Adapter return to Unit Assembly on Page 10.

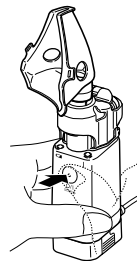
# SELECTING THE NEBULIZATION MODE

This device operates in the Continuous Nebulization Mode or in the Manual Nebulization Mode.

## • Continuous Nebulization Mode

To start the device using the continuous nebulization mode press and hold the ON/OFF Button down for 1 second.

Press the ON/OFF Button again to stop nebulization.


Press and hold the ON/OFF Button down with your finger for 1 second.

## • Manual Nebulization Mode

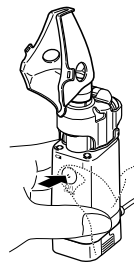In the Manual Nebulization Mode the device will nebulize only when you press and hold the ON/OFF Button down. You can inhale on demand using this mode.

To start the device using manual nebulization press and hold the ON/OFF Button down for at least 2 seconds.

Press and hold the ON/OFF Button to start nebulization.

Press and hold the ON/OFF Button down with your finger for 2 seconds.

**NOTE:** The power indicator (green light) illuminates during nebulization.

# USING THE DEVICE

## **WARNING**

For type, dose, and regimen of medication follow the instructions of your healthcare provider.

## **WARNING**

Always dispose of any remaining medication in the medication cup after each use. Use fresh medication each time you use the device.

## **CAUTION**

Provide close supervision when this device is used by, on, or near infants, children or compromised individuals.

## **CAUTION**

Inspect the Main Unit and the nebulizer parts each time before using the device. Make sure no parts are damaged, the device is assembled properly, and the device operates normally.

## **CAUTION**

Do not operate the device at temperatures greater than +104°F (+40°C).

## **CAUTION**

This device is approved for human use only.

1. Slightly tilt the Main Unit toward yourself to immerse the Vibrating Mesh Cap in the medication.

**NOTE:** If the vibrator is not immersed in the medication, the device will not nebulize.

2. Start inhalation in a relaxed position.
3. Place your lips lightly around the mouthpiece. If using the Child Mask position the mask lightly against the face.
4. Start the treatment as directed by your healthcare provider.
5. Press the ON/OFF Button to turn off the device when finished with your treatment.

**NOTE:** The device has a built-in timer to turn off the power approximately 30 minutes after the power is turned on.

When the AC Adapter is used, remove the power plug from the electrical outlet.

# CLEANING AFTER EACH USE

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying, adhering to the mesh cap, and resulting in the device not nebulizing effectively.

**Wash the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter after each use.**

## **WARNING**

Rinse the nebulizer parts after each use. Dry the parts immediately after washing.

## **CAUTION**

To prevent damage to the device:

- Do not rinse or immerse the Main Unit in any liquid.
- Do not wash or rinse any of the parts under strong running water.
- Do not touch the mesh with your hand or any other object.

1. Remove the Mouthpiece or Mask and the Mask and Mouthpiece Adapter from the Main Unit.
2. Remove the Medication Bottle from the Main Unit.
3. Open the Medication Bottle and discard any remaining medication.
4. Attach the Medication Bottle to the Main Unit. Open the Mesh Cap.
5. Pour a small amount of distilled water into the Medication Bottle and close the Mesh Cap.
6. Turn on the device to nebulize the distilled water for 1 to 2 minutes to remove residual medication from the mesh holes.
7. Turn off the device and remove the Medication Bottle from the Main Unit.
8. Remove the Mesh Cap from the Medication Bottle and discard any remaining distilled water from the Medication bottle.
9. Rinse the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter with distilled water.
10. Gently wipe off excess water with a soft clean cloth or allow the parts to air dry in a clean environment.
11. Assemble the device. Store the device in the Storage Case or in a clean environment.

# DAILY DISINFECTING

**Disinfect the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter after the last treatment of each day.**

## **CAUTION**

To prevent damage to the device:

- Do not rinse or immerse the Main Unit in any liquid.
- Do not wash or rinse any of the parts under strong running water.
- Do not touch the mesh with your hand or any other object.

1. Make a solution using one of the following solutions: Vinegar (1 part white vinegar and 3 parts distilled water), OR mild detergent soap (dishwashing soap in distilled water).

## **CAUTION**

Do not use household bleach. The mesh will rust and the Mesh Cap cannot be used.

2. Lift open the Mesh Cap and pour a small amount of disinfecting solution into the Medication Bottle.
3. Turn on the device to nebulize the disinfecting solution for 1 to 2 minutes.
4. Turn off the device and remove the Medication Bottle from the Main Unit.
5. Remove the Mesh Cap from the Medication Bottle and discard any remaining disinfecting solution from the Medication Bottle.
6. Soak the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter in the disinfecting solution for 10 to 15 minutes.
7. Rinse the Medication Bottle, Mesh Cap, Mask, Mouthpiece, and Mask and Mouthpiece Adapter in distilled water.

## **WARNING**

Do not leave the cleaning solution in the nebulizer parts. Rinse the nebulizer parts with distilled water after disinfecting.

8. Gently wipe off excess water with a soft clean cloth or allow the parts to air dry in a clean environment.
9. Assemble the device. Store the device in the Storage Case or in a clean environment.

# CARING FOR THE DEVICE

To keep your device in the best condition and protect the unit from damage follow these directions:

-  Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed.
-  Store the device and the components in a clean, safe location.
-  Do not use or store the device in humid locations, such as a bathroom. Use the device within the operating temperature and humidity.
-  Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months, or where it will be exposed to direct sunlight.
-  Do not subject the device or any of the components to strong shocks, such as dropping on the floor.
-  Do not disassemble or attempt to repair the device or components.
-  Use only Omron authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
-  Operate the device only as intended. Do not use the device for any other purpose.
-  Changes or modifications not approved by Omron Healthcare will void the user warranty.
-  Dispose of the device, components, and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
-  Remove the batteries if the device will not be used for three months or longer. Always replace all the batteries with new ones at the same time. Do not use different types of batteries together.

# CARING FOR THE DEVICE

## Cleaning the Main Unit

To clean the casing of the main unit moisten a soft cloth with water or a mild detergent. Wipe the casing and immediately dry using a soft clean cloth.

### ⚠ CAUTION

Do not place or attempt to dry the device or any of its parts in a microwave oven.

### ⚠ CAUTION

To prevent damage to the device:

- Do not rinse or immerse the Main Unit in any liquid.
- Do not clean the Main Unit using abrasive cleaners or any type of chemical.
- Do not allow any moisture to contact the electrodes or the AC Adapter Jack on the Main Unit or AC Adapter connection stand.

## Carrying the device in the Storage Case

Assemble the device by attaching the Mesh Cap and the Medication Bottle to the Main Unit. Place the Main Unit Cover on the device.

Position the device in the Storage Case as shown in the illustration.

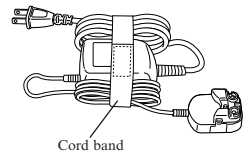
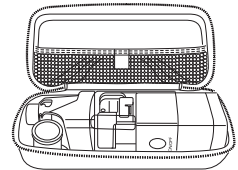
The Mouthpiece and Mask Adapter can be easily placed in the Storage Case to carry with the Main Unit.

### ⚠ CAUTION

To prevent damage to the device

- Do not carry or leave the nebulizer with medication or distilled water in the Medicine Bottle.
- Do not put the AC Adapter in the Storage Case. The Storage Case is not intended to carry the optional AC Adapter.

To carry the AC Adapter bundle the power cord of the AC Adapter and fasten it to the Main Unit of the AC Adapter with the cord band as shown in the illustration.

# TROUBLESHOOTING GUIDE

| PROBLEM   | CAUSE  | SOLUTION  |
|---|--|---|
| <p>The power indicator does not illuminate.</p>                           | <p>The AC Adapter is not plugged into an electrical outlet.</p>  | <p>Turn the power switch off. Plug the power plug into an electrical outlet. Turn the device on.</p>  |
|   | <p>The connection stand is not attached to the Main Unit.</p> <p>The batteries have not been inserted properly. The batteries are low in charge.</p> | <p>Make sure Main Unit has been clicked onto the Connection Stand.</p> <p>Reinsert the batteries.<br/>Replace both worn batteries immediately.<br/>Recharge NiMH batteries with a commercially available charger.</p> |
|   | <p>Residual medication has dried on the components and accessories.</p>  | <p>Clean and disinfect the components and accessories.</p>  |
|   | <p>The mesh cap needs to be replaced.</p>  | <p>Replace the Mesh Cap.</p>  |
| <p>The power indicator is illuminated but the unit does not nebulize.</p> | <p>The batteries are low in charge.</p>  | <p>Replace same type batteries with new alkaline or charged NiMH batteries.</p>   |
|   | <p>The Medication Bottle has too much medication in it.</p>  | <p>Fill the Medication Bottle with the proper amount of prescribed medication. Max is 7 mL.</p>   |
|   | <p>Liquid may have collected around the electrodes of the Main Unit.</p>   | <p>Absorb any moisture with a soft cloth.</p>   |
|   | <p>There is liquid on top of the Mesh Cap.</p>   | <p>Remove visible liquid with a soft cloth and a very light touch so as not to damage the mesh.</p>   |
|   | <p>The medication has not come in contact with the nebulizing parts.</p>   | <p>Slightly slant the Main Unit towards you with the ON/OFF button pointing down.</p>   |

# TROUBLESHOOTING GUIDE

| PROBLEM  | CAUSE   | SOLUTION   |
|--|---|--|
| The unit is on; however, it nebulizes weakly, or is taking too long for treatment. | The Medication Bottle is not properly installed.      | Make sure the Medication Bottle is properly installed.   |
|  | The unit needs to be cleaned.                         | Follow directions for cleaning after each use.           |
|  | The unit needs to be disinfected.                     | Follow directions for disinfecting.                      |
|  | The batteries are low in charge.                      | Replace the batteries per installation instructions.     |
|  | Nebulization rates vary based on the medication used. | Treatment times may vary among medications and patients. |

# WARRANTY INFORMATION

## LIMITED WARRANTIES

Your NE-U22V Nebulizer, excluding the mesh cap and accessories, is warranted to be free from defects in materials and workmanship appearing within 2 years from date of purchase, when used in accordance with the instructions provided with your device. The above warranty extends only to the original retail purchaser.

We will, at our option, repair or replace without charge your Omron Nebulizer. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service, contact Omron Healthcare's Customer Service by calling 1-800-634-4350 for the address of the repair location and the return shipping and handling fee. Information for warranty service is available on our website at [www.omronhealthcare.com](http://www.omronhealthcare.com).

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

**ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.** Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights, which may vary from state to state.

### FOR CUSTOMER SERVICE (US and CANADA)

Visit our web site at:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

# FCC STATEMENT

**Note:**

**POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)**

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)**

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled “Digital Apparatus”, ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

# SPECIFICATIONS

|  |  |
|--|--|
| <b>Model:</b>                              | <b>Omron NE-U22V Nebulizer with V.M.T.®</b>  |
| Power Source:                              | DC 3V AC 120V 60 Hz (with AC adapter)  |
| Power Consumption:                         | 1.5 W  |
| Nebulization Rate:                         | 0.25 mL/min minimum  |
| Particle Size Range:                       | MMD approximately 5µm  |
| Medication Capacity:                       | 7 mL maximum   |
| Operating Temperature/Humidity:            | +50°F to +104°F (+10°C to +40°C) / 30% to 85% RH   |
| Storage Temperature/Humidity/Air Pressure: | -4°F to +140°F (-20°C to +60°C) / 10% to 95% RH / 700hPa to 1060 hPa   |
| Vibration Frequency:                       | 180 kHz  |
| Dimensions:                                | 1.5” (l) x 2.1” (w) x 4.1” (h)<br>(38mm x 51mm x 104mm)  |
| Weight:                                    | 3.4 oz. (97 g)   |
| Battery:                                   | 2 “AA” Alkaline or NiMH Rechargeable (not included)  |
| Contents:                                  | Main unit, medication bottle, carrying case, mouthpiece, mesh cap, mask and mouthpiece adapter, video and instruction manual |
| UPC Code:                                  | 0 73796 45122 6  |

- Subject to technical modification without prior notice

Cascade Impactor Testing at 13 lpm:

|   | <b>Pulmicort®</b> | <b>Intal®</b> | <b>Salbutamol®</b> |
|---|-------------------|---------------|--------------------|
| MMD (micron)                            | 6.76µ             | 6.43µ         | 5.79µ              |
| GSD (geometric standard deviation)      | 2.08              | 2.56          | 2.75               |
| Respirable fraction (%mass 0.52 to 6 µ) | 65.0%             | 66.0%         | 73.4%              |

Treatment time 5 minutes for 2ml.

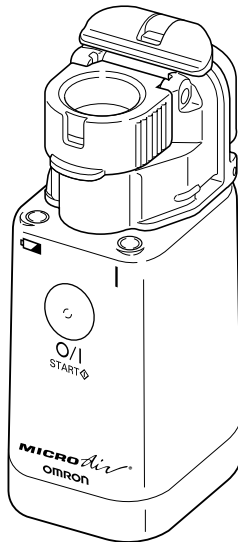
- Please note that specifications may vary with medication type used.

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Copyright © 2007 Omron Healthcare