


## Mirage Liberty™

FULL FACE MASK

### Fitting Guide

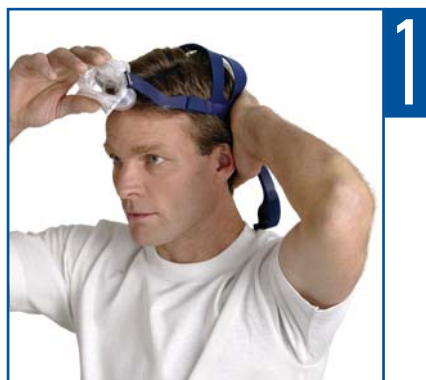

- Place the headgear on the head like a cap and pull the mask forward onto the face.
- Make sure the headgear fits evenly on the top of the head.



- Insert the pillows so that they sit gently at the entrance to the nostrils.
- Fit the mask over the mouth.
- Fasten the lower headgear clips ensuring they click into place.



- Unfasten the Velcro™ tabs and pull the upper headgear straps evenly.
- Do not overtighten the straps.



- Unfasten the Velcro tabs and pull the lower headgear straps evenly.
- Alternate adjustment of upper and lower straps until a comfortable and even fit is obtained.
- Do not overtighten the straps.



- If the pillows are incorrectly positioned at the entrance to the nostrils, readjust them. Press them down into the base and then slowly release, guiding them into position.



- Connect the elbow on the inlet tube to the frame ensuring it clicks into place.
- Rotate the elbow/swivel to ensure it rotates freely on the frame.
- Turn on the flow generator.
- If any air leaks are felt, readjust the pillows (as shown in Step 5) and/or the headgear.

 For details, refer to the *Mirage Liberty User Guide*.