# Porta-neb with SideStream

 **MultiDoctorShop**  
Medical products at home  
We provide the official  
document of the manufacturer  
[www.multidoctorshop.com](http://www.multidoctorshop.com)

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[www.philips.com/respironics](http://www.philips.com/respironics)

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sense and simplicity

## Table of contents

<b>1</b>	<b>Important</b>	<b>3</b>
1.1	Standard compliance	3
<b>2</b>	<b>Your Porta-neb nebulizer compressor system</b>	<b>4</b>
2.1	Parts of the system	4
<b>3</b>	<b>Before you start</b>	<b>4</b>
3.1	Setting up your Porta-neb	4
<b>4</b>	<b>SideStream</b>	<b>5</b>
4.1	Filling the SideStream nebulizer with your medication	5
4.2	Taking your treatment	6
4.3	After your treatment	6
<b>5</b>	<b>Maintenance</b>	<b>7</b>
5.1	Cleaning	7
5.2	Care for your Porta-neb and its air filter	7
<b>6</b>	<b>Accessories and spare parts</b>	<b>7</b>
<b>7</b>	<b>Guarantee and service</b>	<b>8</b>
<b>8</b>	<b>Technical information</b>	<b>9</b>
8.1	Product specification	9
<b>9</b>	<b>Frequently asked questions</b>	<b>10</b>
9.1	If things go wrong	10
	<b>Contact information</b>	<b>10</b>

# 1 Important

The Porta-neb should be connected to a 230 V~ 50 Hz power supply.

Class II  BF 

Compressor	Wobble
Inlet filter	Porous polyethylene
Inlet filter pore size	250 micron

The SideStream drug container is intended for single patient use.

## 1.1 Standard compliance

The Porta-neb portable nebulizer units are manufactured and tested to the requirements of EN60601-1:1990 +A1:1993, A2:1995 & A13:1996; EN60601-1-2:2002.

Compliant with the WEEE/RoHS recycling directives. If you are subject to the WEEE/RoHS directives, contact Philips Respironics for the passport for recycling this product.

## Warnings

- Make sure that the electrical connection is secure.
- When you are using the Porta-neb, be careful to keep the electrical parts away from water.
- Unplug the unit from the power socket:
  - when the treatment is finished;
  - before any cleaning procedure of the unit;
  - in case of failure or malfunction of the unit.
- The unit should not be used in the presence of anaesthetic mixture flammable with air, oxygen or nitrous oxide.
- This is a medical device for aerosol therapy and may only be used with medication prescribed or recommended by a doctor.
- Refer also to the warnings throughout this manual.


## 2 Your Porta-neb nebulizer compressor system

This booklet will help you to understand how to use your Porta-neb SideStream to get the best treatment. If you have any problems, your doctor or nurse will be able to help you.


### **Warning**

Air is drawn into the Porta-neb through the air vents on the sides. Be careful not to cover or block these.

**A**

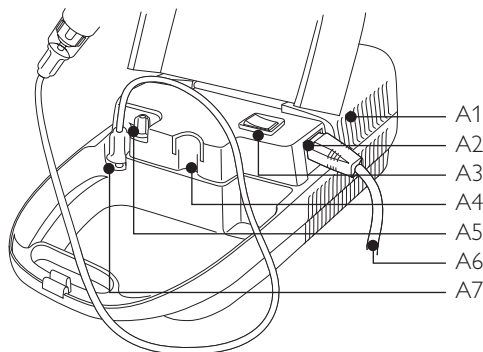**B**
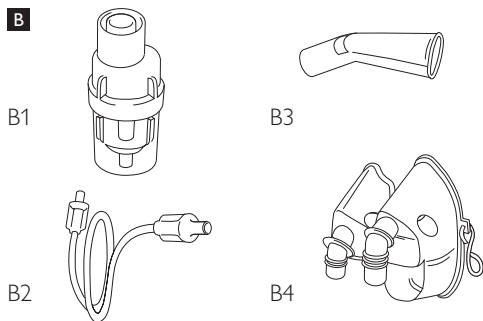
### 2.1 Parts of the system

Diagram A shows the Porta-neb.

- A1 Air vents
- A2 Electrical mains socket
- A3 On/off switch
- A4 Holder (post) for drug container
- A5 Air inlet filter
- A6 Mains lead
- A7 Air flow outlet


Diagram B shows the Porta-neb parts that you will need to take your medication.

- B1 SideStream drug container
- B2 Duratube
- B3 Mouthpiece
- B4 Child mask / adult mask


## 3 Before you start

### 3.1 Setting up your Porta-neb

- 1** Push the mains lead into the mains socket on the side of the Porta-neb, ensuring that it is fully inserted. Then push the 3-pin plug into the mains supply.

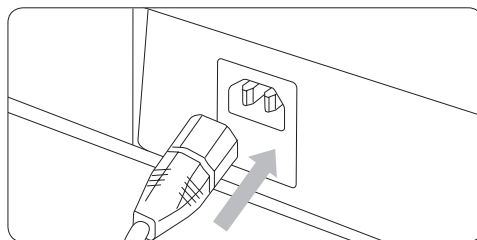
- 2** Next, connect one end of the long, clear tubing (Duratube) onto the air flow outlet on your Porta-neb.

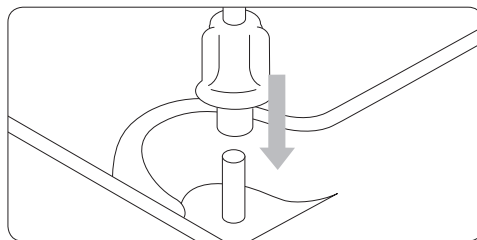- 3** Then, push the other end of the Duratube onto the stem at the bottom of your SideStream drug container.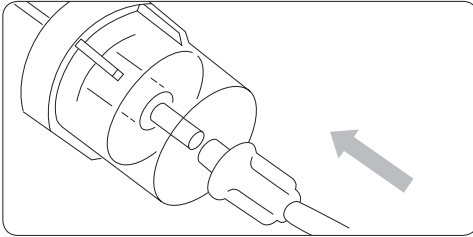


Your Porta-neb is now ready.

## 4 SideStream


### 4.1 Filling the SideStream nebulizer with your medication

Make sure that you have had instructions about the use of your medication.


- 4** Unscrew the base of the SideStream drug container.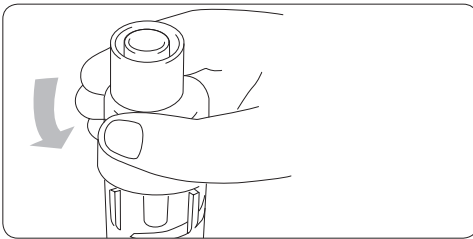

- 5** Put the medication into the base of the SideStream drug container. For more stability when filling medication, use the post on Porta-neb to hold the SideStream nebulizer.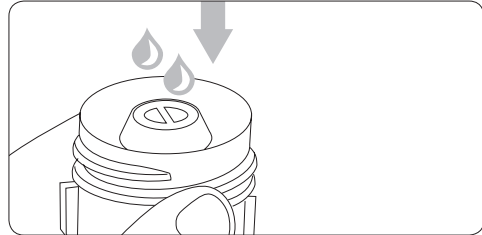


- 6** Screw the top of the SideStream nebulizer back onto the base.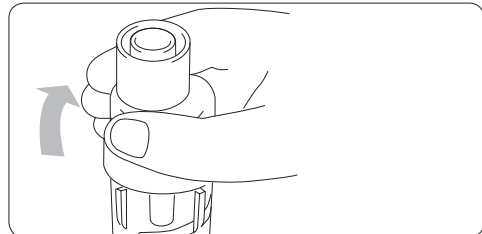

- 7** Connect one end of the Duratube onto the air flow outlet on your Porta-neb.

- 8** Then, push the other end of the Duratube onto the stem at the bottom of your SideStream drug container.

- 9** Now connect either the mouthpiece or the mask to the top of the SideStream drug container.

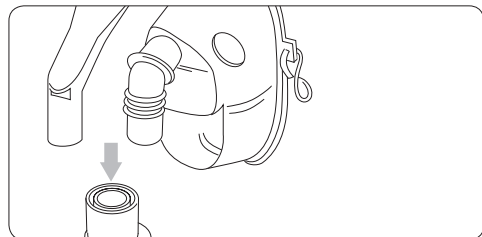The use of a mouthpiece instead of a mask may help to avoid drug aerosol irritating the eyes. Some medications should only be administered using a mouthpiece.

**Note**

If your doctor has told you to dilute the solution, empty the medication in the same way and then add the amount of saline your doctor instructed.

**4.2 Taking your treatment**

You will probably be given help on how to take your treatment by a doctor or nurse.


Remember to relax as much as possible so you get the most benefit from your medication. Sit upright in a chair or in bed.

When you are ready, switch the Porta-neb on at the on/off switch (position **I** is on and **O** is off).

Your treatment will now begin.


**Breathing**

Do not try to breathe quickly. The treatment should not be hard work. If you get breathless during treatment, you may be breathing too quickly. If you want to take a short break during the treatment you can switch off your Porta-neb. Treatment is finished when the SideStream drug container begins to 'spit'.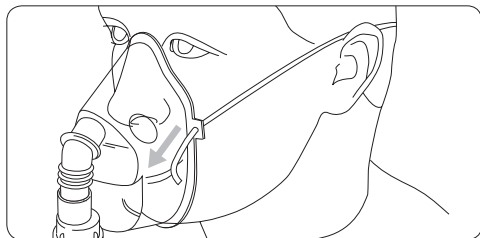


**4.3 After your treatment**

- 10** Switch off the Porta-neb.
- 11** Remove the mask or mouthpiece and Duratube from the SideStream drug container.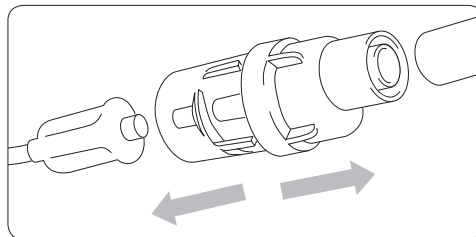

- 12** Unscrew the SideStream drug container and wash the three pieces in warm soapy water, together with the mouthpiece or mask.

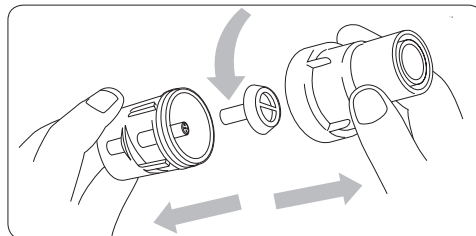- Warning**  
Never put the Duratube in water or any other liquids.

- 13** Reconnect the Duratube onto the SideStream and switch the Porta-neb on for a few seconds to blow out any trapped medication.
- 14** Switch off the Porta-neb and remove the Duratube.



- 15** To prevent the jets from becoming blocked with medication, the SideStream drug container must be boiled once a week, in water with 2-3 drops of liquid detergent for 10 minutes. (Take apart the SideStream drug container before boiling).

**! Cautions**


- When you clean the SideStream drug container be careful not to damage the jet holes (see diagram below).
- Never use a cleaning brush or put paper clips or needles into the jet holes. This will damage the drug container.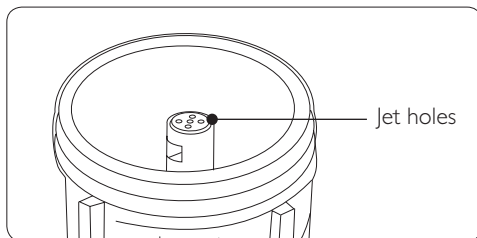

- 16** As soon as the filter no longer looks white and clean it should be changed (usually after three months). Filters will need to be replaced more frequently if you have smokers in your home.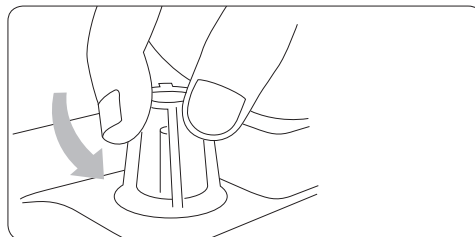

- 17** To change the filter, unscrew the plastic dome, take out the old filter and put in a new one. Then screw the dome back on.

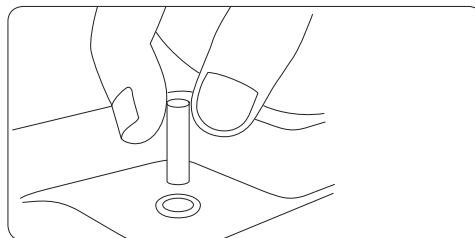## 5 Maintenance

### 5.1 Cleaning

In addition to your routine cleaning after use, the SideStream drug container must be boiled once a week.

**! Warning**

The air filter and the Duratube must not be cleaned in any liquids and should be replaced if dirty.

### 5.2 Care for your Porta-neb and its air filter

Your Porta-neb should be cleaned by wiping the inside and outside of the case with a damp cloth. The filter which cleans the air that you use during treatment, gets dirty over time. If it gets blocked the Porta-neb does not work properly.

## 6 Accessories and spare parts

We recommend that you change the following parts at least once per year:

- SideStream drug container
- Mouthpiece
- Duratube

The air inlet filter should be replaced at least every three months.

If you need advice, telephone customer service. You will need to quote the name of the drug you use and the serial number of your Porta-neb, which you will find inside the lid of your Porta-neb.

Accessories description	Part number
Duratube	2150A
SideStream nebulizer	1200A
Adult aerosol mask	1100A
Child aerosol mask	1120A
Angled mouthpiece	1605A
Air inlet filters (pack of 4)	2153A

Porta-neb accessory year packs	Part number
SideStream year pack - adult mask	2220NF
SideStream year pack - child mask	2221NF
SideStream year pack - mouthpiece	2222NF

## 7 Guarantee and service

### Our commitment to you

The Philips Respironics Porta-neb compressor is guaranteed against faults in materials and workmanship for three years after purchase (equipment must be cleaned and maintained in accordance with the instructions).

If anything should go wrong with your Porta-neb compressor within three years from date of purchase, simply contact Philips Respironics who will advise you how to return the Porta-neb for repair.

The Philips Respironics SideStream drug container is guaranteed for 12 months from the date of purchase, based on 4 x 6 minute treatments per day, i.e. 1460 treatments (the SideStream drug container must be cleaned as shown in the instructions).

Charges will be incurred if:

- a) The Porta-neb SideStream system is returned and no fault is found.
- b) Damage to the Porta-neb SideStream system has been caused by the user, e.g. Porta-neb compressor has been dropped, drugs/water etc. spilt into the motor or someone has tampered with it.

The remainder of the system which consists of the Duratube, mouth-piece and masks are not covered by these guarantees. This is because these items are low-cost, limited-life components and due to individual handling, cleaning and sterilisation, it is not possible to accurately predict the life of these items.

### Your commitment to us

To regularly clean and maintain the Porta-neb compressor and SideStream drug container in accordance with the instructions. Ensure that the Porta-neb is serviced only by the qualified engineers, using Philips Respironics approved parts.

Only genuine Philips Respironics parts should be used within your Porta-neb SideStream system and all system parts are replaced at least once a year. Failure to maintain your system and replace critical parts such as the air filters (4 times a year) and SideStream drug container (once a year) may adversely affect your treatment and condition.

This guarantee does not affect your statutory rights.

### Service

Servicing on all Philips Respironics compressors can be carried out by the following engineers using the current Philips Respironics service manual and the recommended test equipment:

- Engineers trained and certified by Philips Respironics.
- Hospital EBME engineers who have gained an Electrical or Mechanical ONC qualification.

Only Philips Respironics approved parts may be used to service this equipment. Use of non-approved parts may invalidate the compressor's test certificate and warranty.

We recommend you have your Porta-neb serviced every 12 months by Philips Respironics or a suitably qualified engineer.

## 8 Technical information

### 8.1 Product specification

#### Voltage

- 230 V~ 50 Hz

#### Power

- 210 VA

#### Fuse (in unit)

- T2 A / 250 V

#### Approximate dynamic pressure & flow\*


- 7 L/min at 1 bar

#### Weight

- 2.9 kg

#### Dimensions

- L = 380 mm, W = 220 mm, D = 120 mm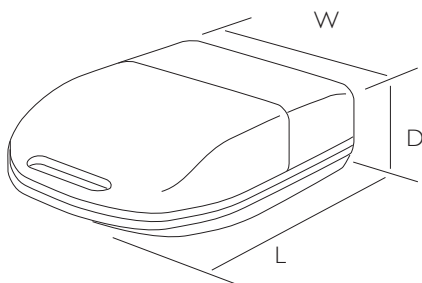

#### Operating conditions:

##### Temperature

- Min. 10°C; max. 40°C

##### Air humidity

- Min. 30%; max. 75%

##### Atmospheric pressure

- Min. 700 hPa; max. 1060 hPa

#### Storage conditions:

##### Temperature

- Min. -25°C; max. 70°C

##### Air humidity

- Min. 10%; max. 95%


##### Atmospheric pressure

- Min. 700 hPa; max. 1060 hPa

\* Pressure at the nebulizer jet and flow through the nebulizer jet.

#### Fuse

You may need to change the fuse (T2 A / 250 V) in the Porta-neb. This can be found in the mains socket on the side of the Porta-neb. Use a small screwdriver to release the panel and then change the fuse (see diagram below).

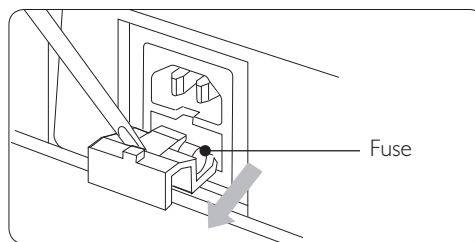#### Servicing manual/wiring diagrams

Full service manual/wiring diagrams are available from Philips Respironics to help qualified people service the Porta-neb (part no. 1071642).

## 9 Frequently asked questions

## Contact information

### 9.1 If things go wrong

**Unit does not run. The main switch on top of the Porta-neb does not light.**

Possible solution: Push mains lead firmly into socket. Replace the fuse on the Porta-neb in the mains lead socket.

**The switch lights up but the Porta-neb runs slowly.**

Possible solution: Check filter - part no. 2153A

**The switch lights up but the Porta-neb does not run.**

Possible solution: Return the Porta-neb for service.

**Unit runs but SideStream drug container makes little mist.**

Possible solution: Clean the SideStream drug container by boiling in water with 2 drops of liquid detergent for 10 minutes. Do not use a cleaning brush. You may need to buy a new SideStream drug container.

**SideStream drug container bubbles but makes no mist.**

Possible solution: Take the SideStream drug container to pieces and clean as explained above.

For any queries relating to the use of your Porta-neb, contact customer service:

Tel: +1 724 387 4000

Fax: +1 724 387 5012

Email: [globalcustomerservice@respironics.com](mailto:globalcustomerservice@respironics.com)

CE 0123

Respironics New Jersey, Inc.  
5 Wood Hollow Road  
Parsippany, New Jersey, 07054 USA

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All rights reserved.
Respironics Deutschland GmbH